



CLONMED™ CAUTPEN

CAUTERY PENCIL



JUPITER HEALTH CARE

SHOP NO 83 SECOND FLOOR OMAXE COMMERCIAL TOWER SECTOR-8
SONIPAT HARYANA

9999660082 
jupiterhealthcare.t@gmail.com 

Electrosurgical pencils, also known as **CAUTERY PENCILS** are used during electrosurgery procedures. They are used to cut biological tissue and control bleeding through Radio Frequency Alternating Current (RFAC).

Instruction of use:-

- Use aseptic technique when removing a single use ESU pencil from pouch.
- Check electrode connections for secure fit. If electrode is changes. Ensure that the electrode is fully inserted. Remove protective sleeve from electrode.
- Plug the pencil connector into the socket of the Electrosurgical Generator.
- To active cut mode. Depress CUT switch. To activate coag mode, depress COAG switch. Ensure that responds correctly.
- Please check the expiration date before use.

Caution:-

- Sterility is guaranteed unless package is damaged or seal is broken.
- This is monopole active instrument. Before use, a dispersive electrode must be properly applied to patient.
- Production and expiration dates are on the bag.